



### **PENETRATION SITE:**

From Tibial Tuberosity (A) go approx. 2 cm toward inner leg (B) and approx. 1 cm toward knee (C)

Hold firmly the bottom part, at 90° to the leg



2

Squeeze and pull out the safety latch



3

While holding firmly at the bottom part, press down with the palm of your hand



4

Pull up the BIG slowly



5

Remove the trocar needle



6

Secure with the safety latch



7

Flush with 10-20 cc of normal saline



8

**Use aseptic technique throughout!**

## **How to use the Adult BIG?**

Brief Guidelines

## **What if...**

**1** Trocar Needle is stuck after triggering?

**Reason:** Inaccurate penetration site.

**WHAT TO DO?**

**Match** safety latch square hole to the hub.

**Pull** needle out gently.

**Find** the right penetration site on the other leg.

**Use** a new BIG.

**2** Needle is not stable?

**Reason:** Unfirm hold and/or wrong angle positioning.

**WHAT TO DO?**

**Pull** needle out.

**Find** the right penetration site on the other leg.

**Use** a new BIG, while holding firmly at 90° to the leg.

**3** Needle is in place, but there is no flow?

**Reason:** cannula is blocked.

**WHAT TO DO?**

**Flush** with a syringe up to 20cc of normal saline and proceed with infusion.

rev. 10/06



WaisMed Ltd.  
Leading Intraosseous Devices

[www.waismed.com](http://www.waismed.com)

[mail@waismed.com](mailto:mail@waismed.com)